
Chitosan Dressing Instructions

【Product Name】 Chitosan Dressing

【Models and Specifications】 Chitosan dressings are available in 5g/patch, 7g/patch, 15g/patch, 20g/patch, 30g/patch, 35g/patch, and 50g/patch sizes, with a tolerance of $\pm 9\%$. Special specifications are available upon request.

【Product Performance, Structure, and Composition】 Chitosan dressings consist of a non-woven backing layer and a dressing. The dressing is composed of purified water, carbomer, and trehalose; the non-woven backing layer is a blend of chitosan fibers and non-woven fabric. The ingredients in the chitosan dressing are not absorbable by the human body and are sterilized by irradiation. The product is supplied sterile.

【Key Performance Indicators】

1. The cytotoxicity of the materials used in the product should not exceed Grade II;
2. The product should not exhibit delayed hypersensitivity reactions or intradermal irritation;
3. Sterility: Sterile after irradiation sterilization.

【Scope of Application】 For covering and caring for non-chronic wounds. [Contraindications]

1. Use with caution if you have allergies.
2. Do not use with pregnant or lactating women.
3. Use with caution with children.

【Instructions】 Remove the clip and push the dressing in the direction of the arrow onto the non-woven medical cloth, allowing it to absorb completely and evenly. Remove the non-woven medical cloth and unfold it. Apply to the face for 15-20 minutes each time, once daily, or as directed by a physician.

【Precautions】

1. Do not use if the product has expired.
2. This product is sterile. Please carefully inspect the inner packaging before use. Do not use if it is damaged or contaminated.
3. This product is for single use only. Reuse is strictly prohibited. Dispose of after use.
4. Please read the instructions carefully before use.



One-time use, destroyed after use



Do not use if the packaging is damaged



Sterilized by irradiation

Storage Conditions and Methods: Store indoors in a ventilated, dry, light-proof environment with a relative humidity not exceeding 80%, free from corrosive gases.

【Registrant/Manufacturer】 Changde Yinuo Biopharmaceutical Co., Ltd.

【Registrant/Manufacturer Address/Manufacturing Address】 Fengkang Avenue, Xicheng

Community, Dajinggang Town, Anxiang County, Changde City, Hunan Province (within Jinggang Park)

【After-Sales Service】 Changde YINUO Biopharmaceutical Co., Ltd.

【Production License Number】 Hunan Food and Drug Administration Medical Device Production Permit No. 20210068

【Registration Certificate Number/Product Technical Requirements Number】 Hunan Medical Device Registration Permit No. 20212141673

【Manufacture Date】 See outer box for details

【Product Expiration Date】 2 years

【Product Instruction Manual Revision Date】 April 27, 2021

Note: Products without the Changde YINUO Biopharmaceutical Co., Ltd. traceability code are counterfeit and substandard products.